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Sodio Citrato 46,7%

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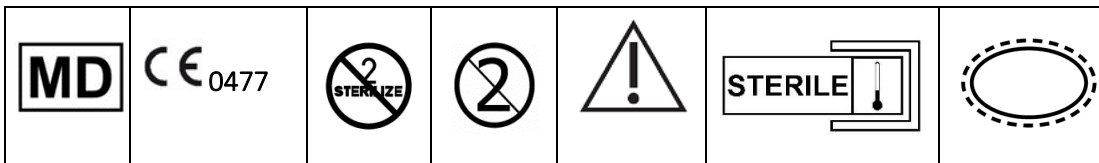
Sodium Citrate 46,7%

Catheter-lock

Anti-clotting, Anti-microbial, Biofilm reduction

COMPOSITION

46,7% Trisodium Citrate in water. The **Sodium Citrate 46,7%** is delivered sterile and supplied as a clear solution. The pH is adjusted to approximately 6,7. Aseptic technique must be maintained at all times. Each single use syringe contains 2,5 ml.

PRESCRIPTION

Sodium Citrate 46,7% is indicated as a catheter lock solution to prevent coagulation of the blood and infection in any type of intravenous catheter.

To prevent:

- Clotting of blood in catheter
- Bacterial infections in catheter

And reduction of:

- Biofilm formation in catheter

PERFORMANCE

Trisodium Citrate (TSC) causes anticoagulation by chelation of ionized calcium in the extracorporeal circuit into a soluble complex. Because calcium is an integral ion involved in the clotting cascade, local removal by citrate prevents the activation of clotting cofactors, factor X, and prothrombin, and the ultimate formation of fibrin. However, systemic anticoagulation will not occur when the ionized calcium concentration will be restored. During metabolism of citrate into bicarbonate, calcium will be released. In this way the level of serum bicarbonate will be maintained.

Antimicrobial effect and a reduction of biofilm formation through binding and removal of Ca²⁺ in the surrounding milieu. Ca²⁺ regulates several genes responsible for growth and multiplying of microbes. Chelation of Mg²⁺ results in instability of the cell wall of microbes.

WARNINGS

- Do not use the syringe if the pouch is damaged
- To be used by skilled and trained personnel
- Dispose unused portions
- **Sodium Citrate 46,7%** may not be applied by either direct intravenous injection
- or added to an infusion
- **Sodium Citrate 46,7%** is intended only for use as Catheter Lock
- Only inject **Sodium Citrate 46,7%** if the exact catheter volume is known (see instructions catheter manufacturer)

SIDE-EFFECTS

- No side-effects known, when the solution is used as catheter-lock.
- In the event that **Sodium Citrate 46,7%** unintentionally passes into the vein, paraesthesia and/or dysgeusia may occur.

INSTRUCTIONS FOR USE

USE ASEPTIC TECHNIQUE

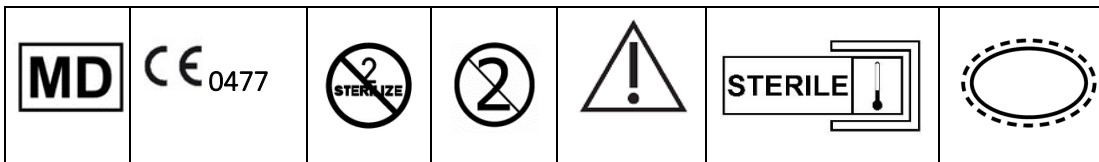
1. Prior to initiation of the venous access session, aspirate and discard the solution that was instilled in the Vascular Access Device (VAD) during the previous session.
2. At the completion of each venous access session, flush the lumens of the VAD with sterile 0.9% Sodium Chloride USP solution to remove any blood, medication or other substance remaining in the VAD.

3. Aseptically open package containing Sodium Citrate 46,7% pre-filled syringes. Inspect the contents of the syringe(s) according to WARNINGS AND PRECAUTIONS.
4. Remove and discard syringe cap. Expel air from syringe. Do not allow air to be trapped in fluid path. Connect the syringe to the VAD. Use only with compatible luer lock connectors.
5. Slowly, over a 5 to 10 second period, instill the exact priming volume of Sodium Citrate 46,7% solution into each VAD lumen, as a locking agent, in volumes corresponding to the luminal capacity as specified by the VAD manufacturer.
6. Remove and discard unused portions and empty syringes according to the institution's bio hazardous waste policy.

Note that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

S.A.L.F. S.p.A. Laboratorio Farmacologico via Marconi, 2 24069 Cenate Sotto (Bergamo) – Italy

Date of revision: 11/2024